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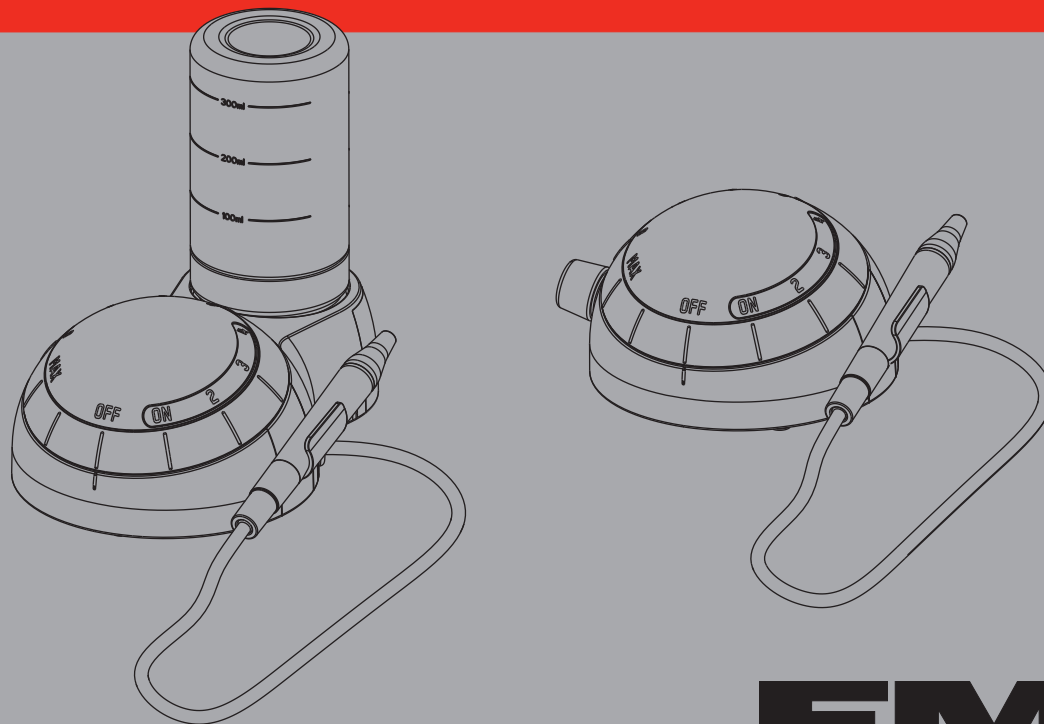
PM100  
PM200

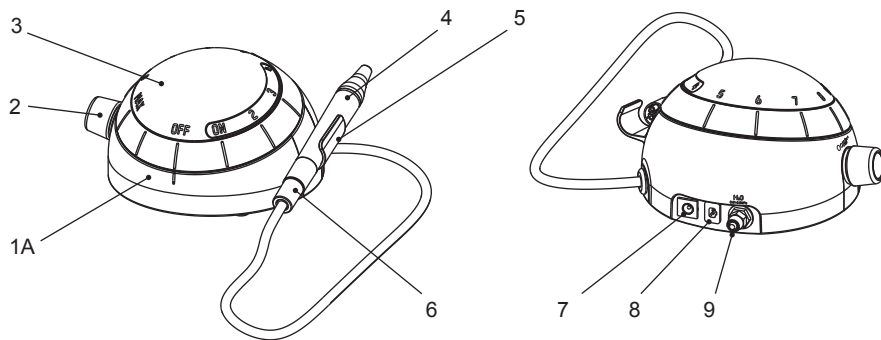
**Rx ONLY** Caution! Federal (USA) law restricts this device to sale by or on the order of a dentist.

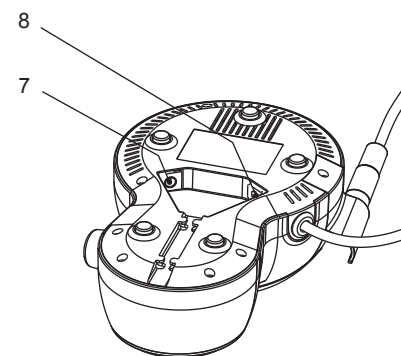
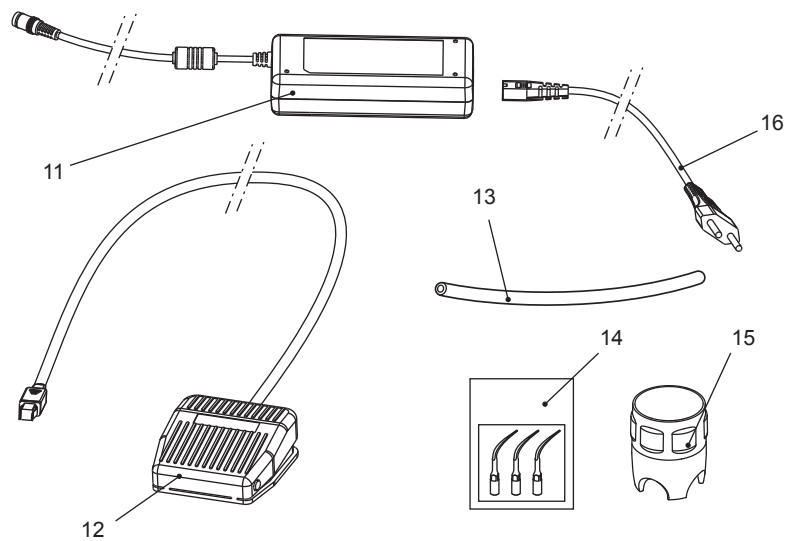
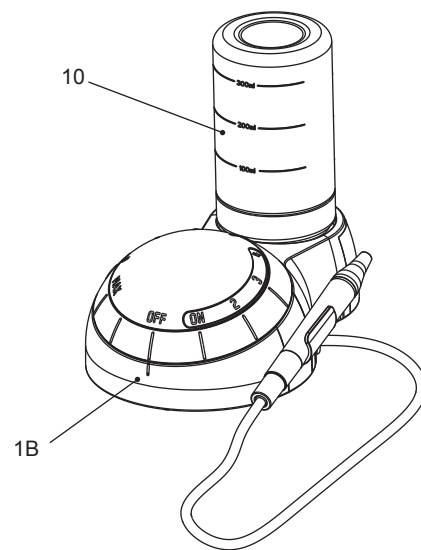
**EMS**   
ELECTRO MEDICAL SYSTEMS

PM100

PM200

## COMPONENTS

- 1 Unit  
1A PM100  
1B PM200
- 2 Water flow rate adjustment
- 3 Power knob
- 4 Handpiece
- 5 Handpiece holder
- 6 Connector of the handpiece cord
- 7 External power supply connection
- 8 Foot switch connection
- 9 Water hose connection
- 10 Bottle
- 11 Power supply
- 12 Foot switch
- 13 Water supply hose
- 14 Set of scaling tips
- 15 Combitorque
- 16 Power cord

EMS supplies units with various accessories. The "Packing list" shows exactly what is included with your unit.

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## DEAR CUSTOMERS,

Thank you for buying a new EMS product. This device meets the highest quality and safety standards.

### Indications

This product is designed for use in dentistry with EMS ultrasound instruments for the following indications:

#### Scaling

- Removal of supragingival calculus
- Removal of stains

#### Endo

- Preparation, cleaning and irrigation of root canals
- Retrograde preparation of root canals
- Condensing gutta-percha
- Removal of crowns, bridges and restorations

For further details on the various instruments and their scopes of application, please refer to the operation instructions supplied with your Piezon system or available from the distributor in your country.

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## ABOUT THIS MANUAL

 The illustrations in these instructions represent the PM100 (unless otherwise specified); however, these instructions also apply to the PM200.

These operating instructions are to ensure the correct installation and use of this product. Always keep this document close at hand.

Please read these operating instructions carefully as they explain the most important details and procedures. Please pay special attention to the safety precautions.

Instructions concerning the components of the system are provided separately in the relative instructions for use. Read the instruction manuals originally delivered with each of the components before using this product. Do not hesitate to contact your local EMS dealer for further information.

This product needs special precautions regarding electromagnetic compatibility and needs to be installed and put into service according to the "Electromagnetic compatibility" manual.

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To prevent possible patient or user injury or damage to the PM100 and 200, please note the corresponding directives:

**Danger**

Risk of patient or user injury

**Caution**

Risk of damage to the device or environmental harm

**Please note**

Useful additional information and hints

**Prohibited****Authorized**

Note that the English version of this manual is the master from which the translations derive. In case of any discrepancy, the binding version is the English text.

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call your approved EMS repair center or your dealer directly.

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EMS

## CONTENTS

Installation and connection.....	2
Connecting the unit .....	2
Electrical Safety precautions .....	6
Preparing for treatment.....	8
Mode and instruments.....	8
Connecting the handpiece .....	10
Attaching the instruments .....	12
Installing the bottle (PM200).....	14
Solutions compatible with the circuit .....	16
Operating the unit .....	18
Setting the ultrasound power .....	20
"Endo" mode.....	22
"Scaling" mode .....	24
Setting the liquid flow rate .....	26
Foot switch.....	28
End of treatment .....	30
Circuit washing .....	32
Handpiece disassembly .....	34
Cleaning and sterilizing .....	36
Cleaning.....	36
Packaging for Sterilization .....	38
Sterilization.....	38
Cleaning the bottle and the unit .....	40
Disinfecting the circuit .....	42
Maintenance .....	44
O-rings checking .....	46
Replacing the pump and the filter .....	48
Safety precautions.....	50
Storing the product .....	54
Disposal .....	56
Warranty.....	56
Accessories .....	56
EMS Service .....	58
Symbols .....	60
Technical data.....	66
Troubleshooting .....	70

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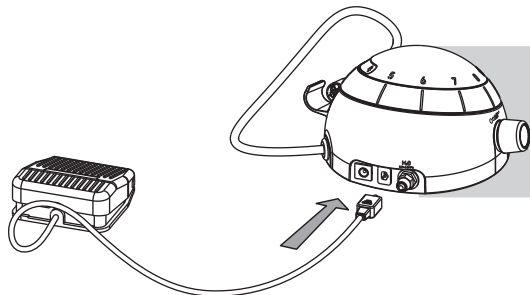
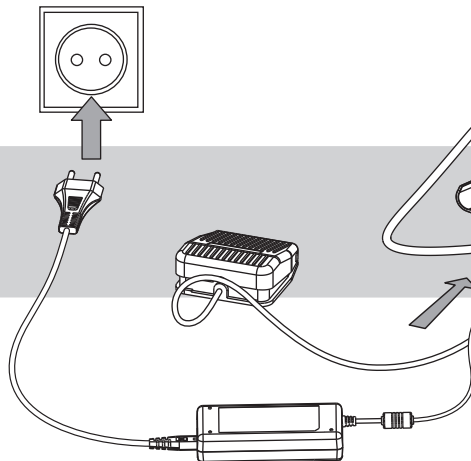
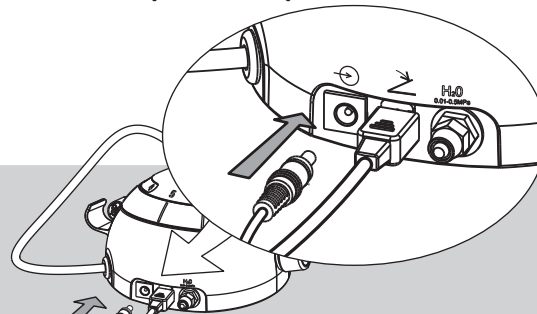
 Always check that the unit is installed in a safe and stable position. A possible fall may suddenly pull on the cord and the handpiece, hurting the patient and the user, and damaging the unit.

 Check that the rated voltage of the power supply corresponds to the local 661 line voltage to prevent damaging the unit.

Use the unit only in a FI protected mains supply (FI = Residual current protection).

 For USA or Canada: connect only to a Hospital Grade outlet.

Connect the unit to a water supply with the original EMS water hose supplied with your product.

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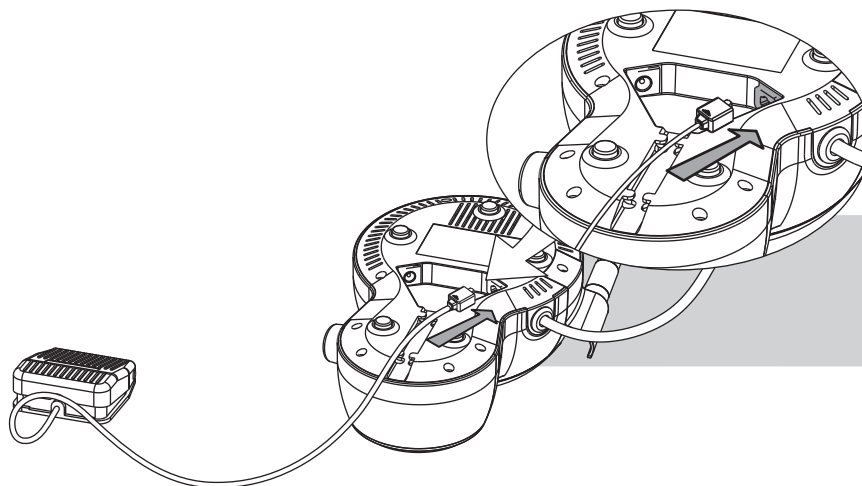
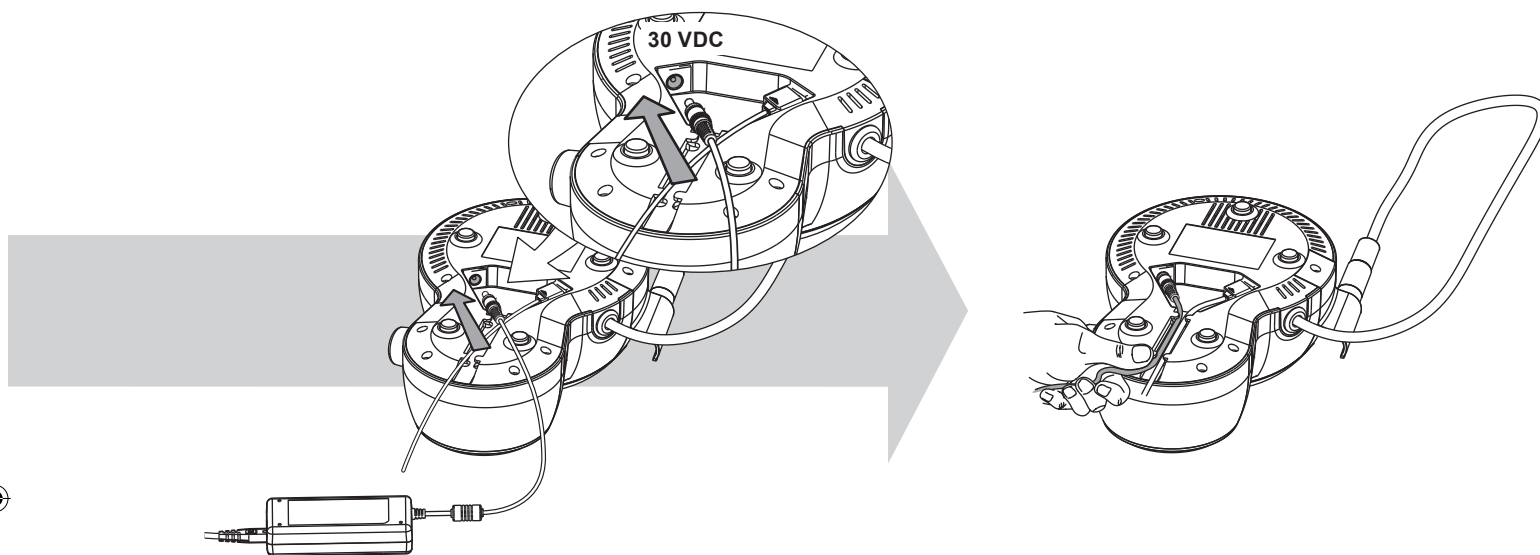
To connect the foot switch and the power supply to the PM200, turn over the unit as shown in the pictures above.

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## Electrical Safety precautions

 No modification of the unit and the accessories supplied with the unit are allowed.

 Do not touch the connector of the handpiece cord and the patient simultaneously.

 The power supply must not be fixed tightly by screw.

The power supply can be placed on a table or hung on the wall. So it is accessible and can easily be detached.

 To disconnect the unit from the supply network, unplug the power supply.

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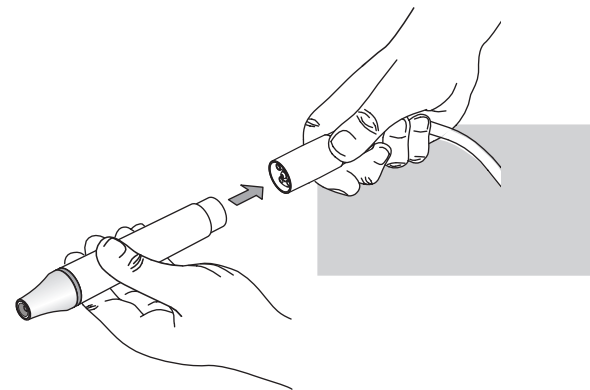
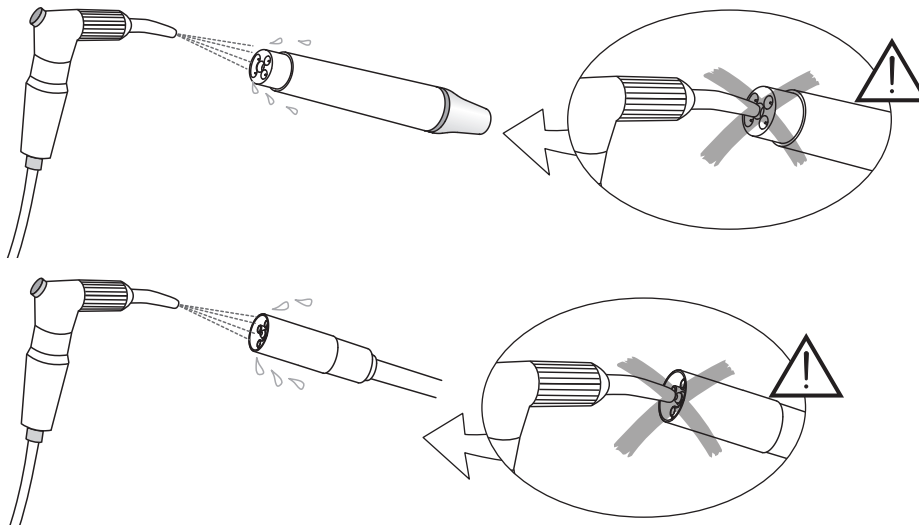
 Clean, disinfect and sterilize the different pieces and accessories of this product before each use. Please refer to the "Cleaning, disinfecting and sterilizing" chapter. Non-sterile pieces and accessories may cause bacterial or viral infections.

 Blow dry the connections to remove the possible presence of liquid to ensure a proper electrical contact. Never blow compressed air in the irrigation connections as this will irretrievably damage internal parts.

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## Solutions compatible with the circuit

 Please read the instructions for use of the disinfecting products.

 Please refer to standards applicable in your country regarding the purchase and use of these products.

### Disinfecting solutions for the treatment

- Chlorhexidine ( $\leq 0.3\%$ )
- Physiological water ( $\text{NaCl} \leq 0.9\%$ )
- Hydrogen peroxide ( $\text{H}_2\text{O}_2 \leq 3\%$ )
- Carbanilides
- Quaternary ammonium compounds
- Citric acid
- Aromatic essence
- Ethanol

### Examples of common products

BacterX<sup>®</sup> pro / Triclosan / Listerine<sup>®</sup> / Alodont<sup>®</sup> / Meridol<sup>®</sup> / BreathRx<sup>®</sup> / CloSYS<sup>™</sup> / Scope<sup>®</sup> / Cepacol<sup>®</sup> / .....

### Disinfecting solutions for washing the circuit

- EMS Ultra Clean ( $\text{H}_2\text{O}_2$  6%)
- Hydrogen peroxide ( $\text{H}_2\text{O}_2$  5%)
- Sterilox (hypochlorous acid)

### Incompatible mixing

 Different disinfecting solutions can be used during treatment. Chemical reactions might occur between these solutions and damage the unit. Wash the liquid circuit for 20 seconds with distilled (or demineralized) water before using a different disinfecting solution (please refer to the "End of treatment" chapter).

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 The "Scaling" mode uses the entire power range for the ultrasounds, from "ON" to "MAX".

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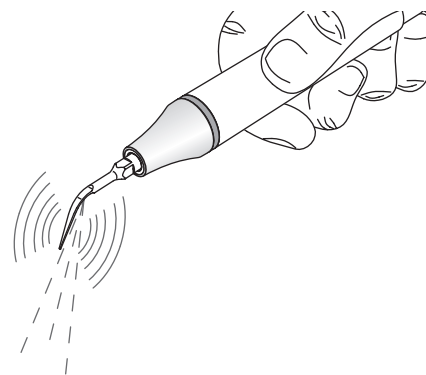
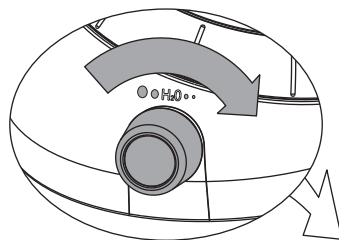
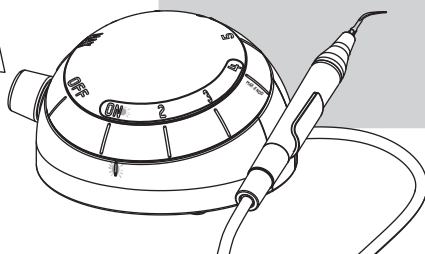
 Optimal settings for the liquid flow rate for each instrument are indicated in the operating instructions of the Piezon systems.

 Never work dry to avoid heat damage on the tooth, unless the instrument is intended specifically for this purpose. The tip of the instrument immediately heats up when used dry.

 For further information concerning instruments usable in "DRY WORK" mode, please refer to the Piezon systems manual.

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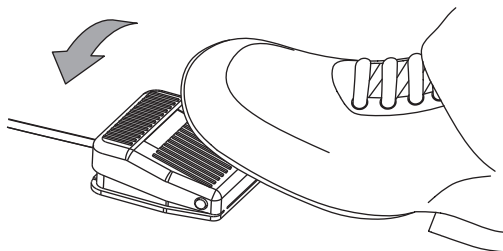
## Pedał nożny

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 Upon first use, keep the pressure on the foot switch until the liquid comes out of the handpiece.

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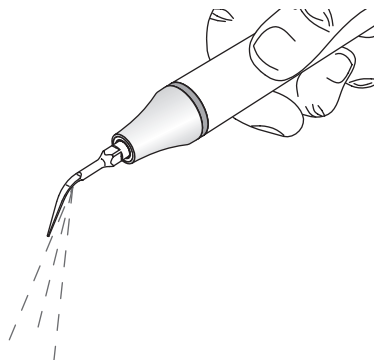
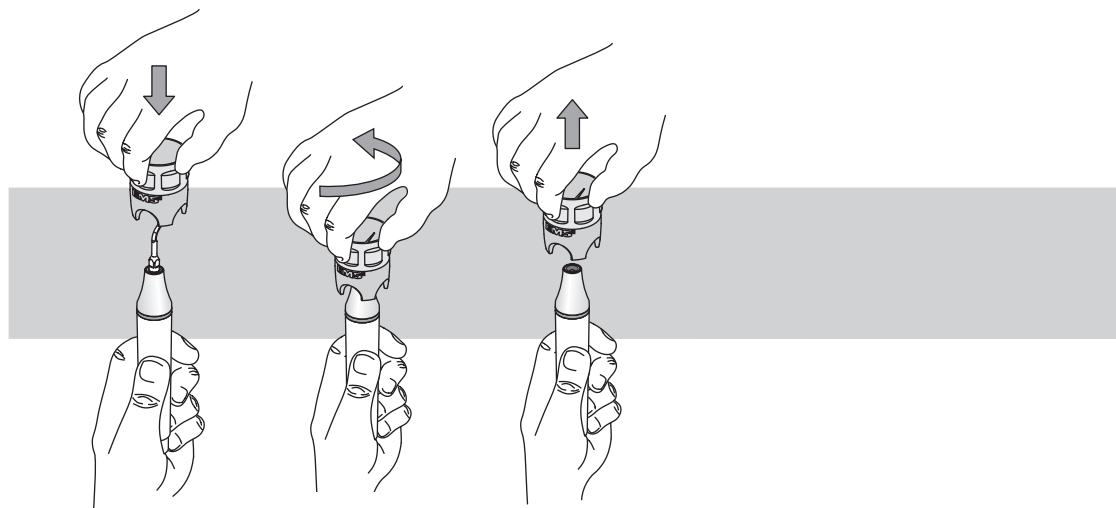
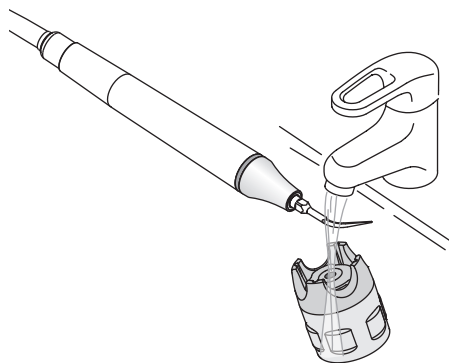
 Clean the instrument and the Combitorque with running water before disinfection.

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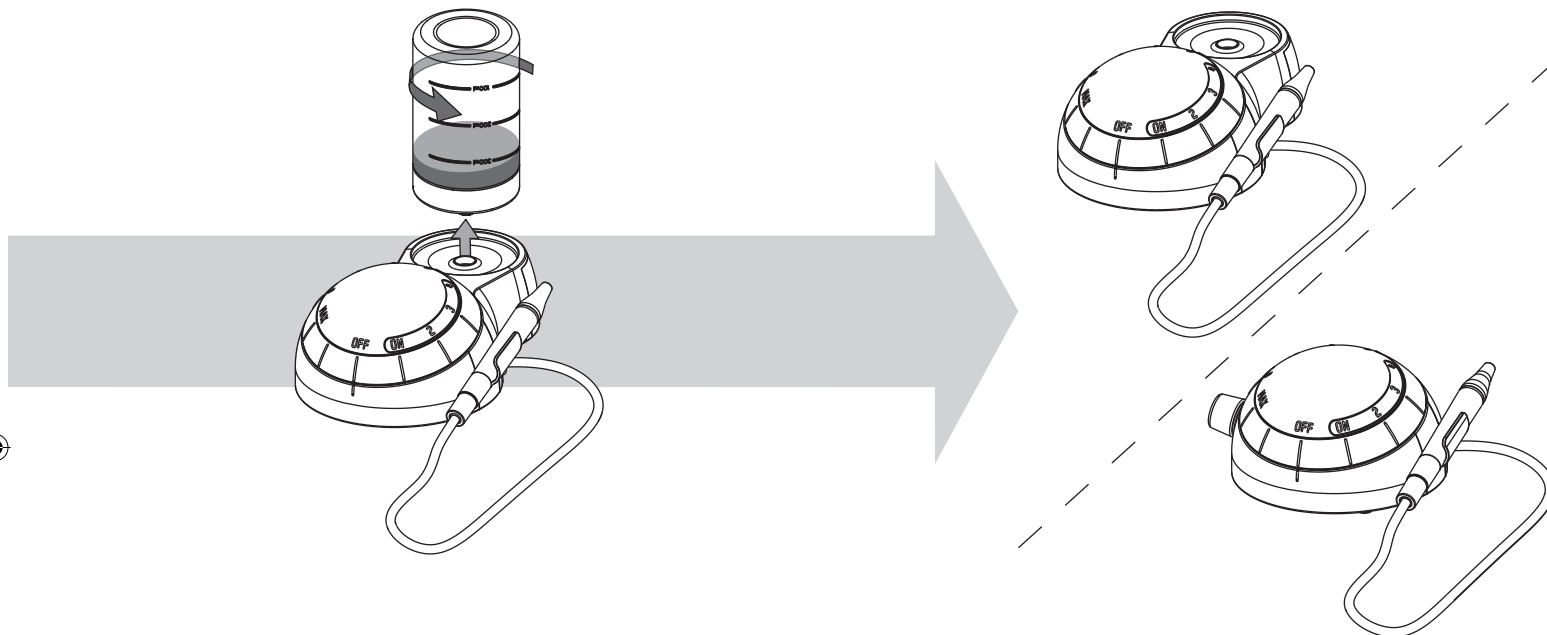
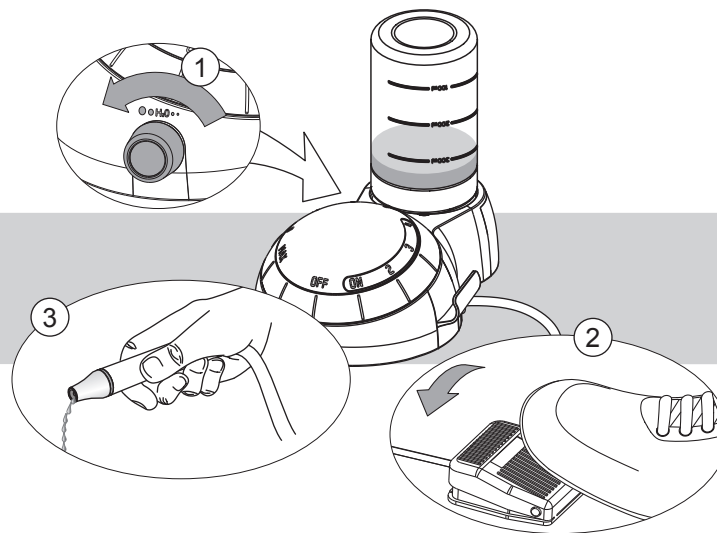
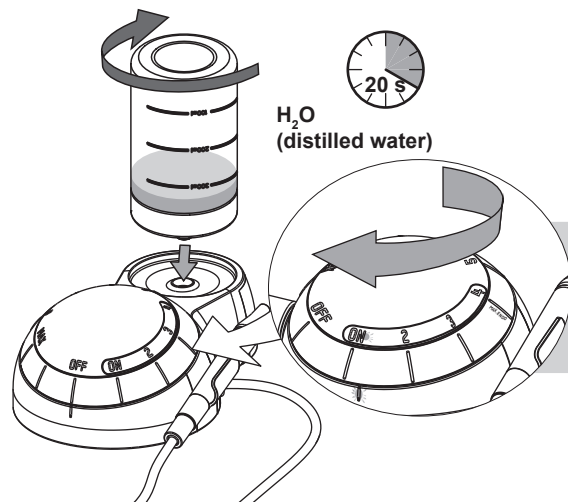
⚠ Once the treatment has been completed, wash the circuit during 20 sec. of distilled (or demineralized) water to avoid a possible reaction caused by ageing of the solution or by an unwanted mixture of liquids.

👉 Clean the unit and the accessories (please refer to the "Cleaning, disinfecting and sterilizing" chapter).

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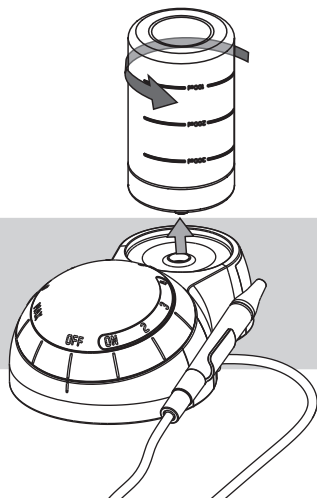
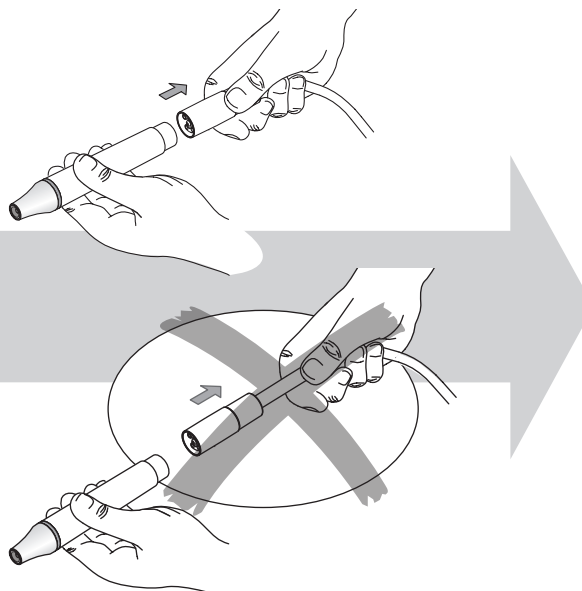
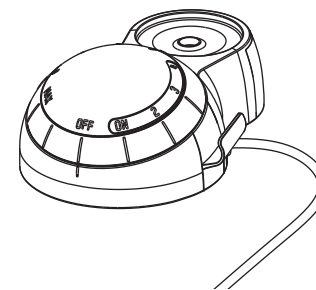
 Unscrew the cap and the light guide (when equipped) under running water before cleaning.

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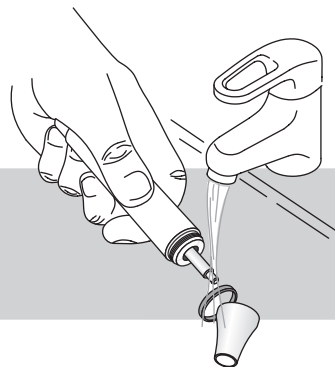
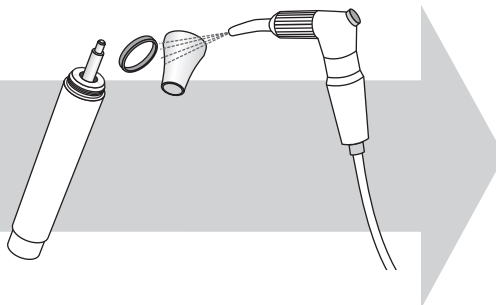
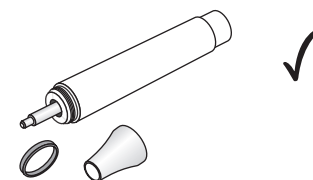
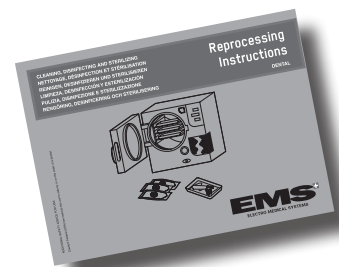
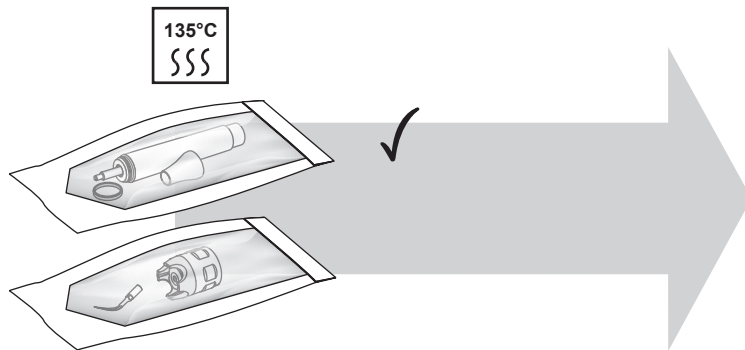
Recommended cleaning, packaging for sterilization, and sterilization procedures are provided below.

### Cleaning

**!** Only the manual reprocessing is validated for the handpiece, the Combitorque and instruments.

- Remove any externally attached soiling by brushing carefully with a soft surface brush or cloth.
- Remove the instruments from handpieces.
- Soak all parts in the cleaning solution for, at a minimum, the time and the concentration specified by the manufacturer of the cleaning agent.

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- Remove any externally attached soiling by brushing carefully with a soft surface brush or cloth.
- Place all parts in an ultrasound bath filled with cleaning solution.
- Treat the parts with ultrasound for at least 3 minutes.
- Brush the parts in the cleaning solution if any visible soiling is detected.
- Submerge all components completely in deionized water. Rinse through all product lumens with deionized water.
- Repeat the cleaning process if the last rinse does not run clear, or if stains are still visible on the product.
- Dry all components at room temperature.

## Packaging for Sterilization

The clean, dry instruments should be placed in a sterilization container. The container should be bagged in a sterilization pouch that is validated for use in a 135 °C steam sterilization cycle. The sterilization pouch should be sealed according to the manufacturer's instructions.

## Sterilization

EMS recommends sterilization in a fractional cycle at 135 °C for at least 3 min. EMS recommends use of biological indicators and chemical indicators that have been validated for use in a 135 °C steam sterilization cycle.

 The hot-air sterilization and radio-sterilization procedure may not be used (as it causes the destruction of products).

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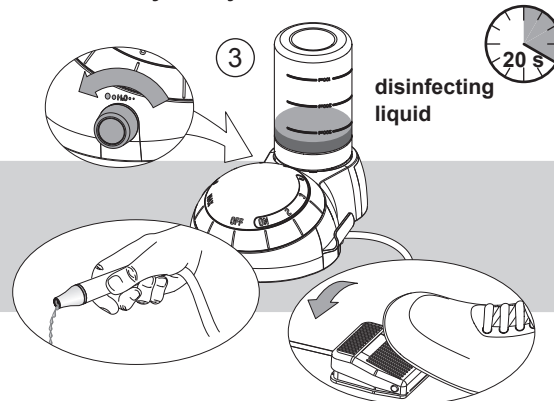
 It is recommended to disinfect the liquid circuit daily.

 Disinfection is necessary to reduce crystalline deposits and the number of bacteria in the liquid circuit of the unit; the following procedure achieves a 5 log reduction of target pathogens (using disinfecting solutions for washing the circuit as listed at the beginning of this manual):

- 1 Remove the instrument,
- 2 Wash the circuit during 20 seconds with distilled (or demineralized) water,
- 3 Disinfect the circuit during 20 seconds with an appropriate disinfecting solution for washing the circuit
- 4 Allow the disinfecting liquid to act for 5 minutes,
- 5 Rinse the circuit during 20 seconds with sterile water.

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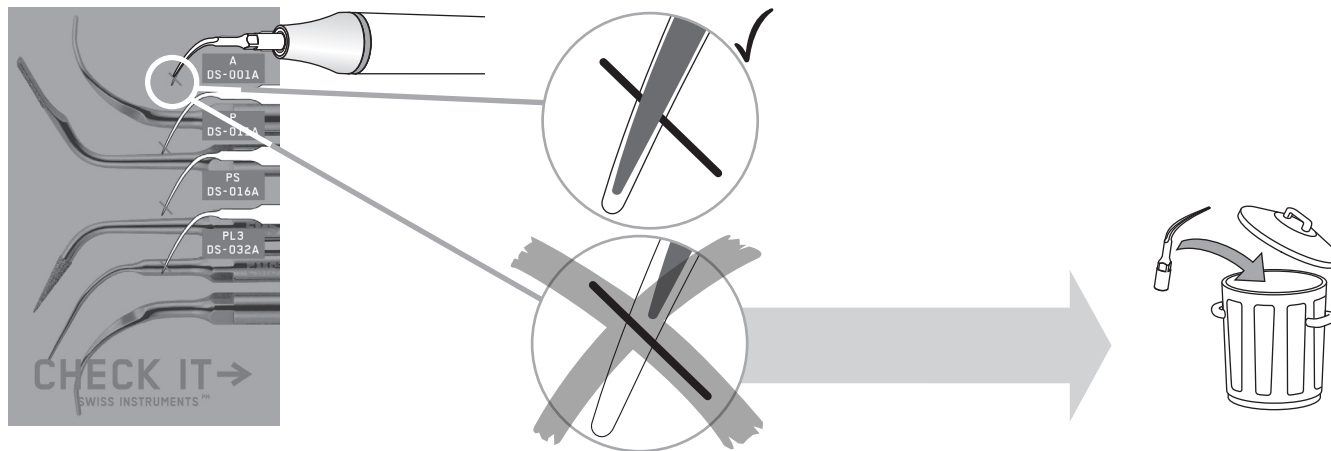
## MAINTENANCE

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## KONSERWACJA

The EMS Piezon instruments must be checked regularly using the EMS "Tip Card". Ultrasound instruments wear during use and become shorter. Worn instruments are ineffective and are a cause of discomfort to the patient.

 Instruments showing wear on the diamond coating must be replaced.

 For precautionary reasons, do not exceed the validated lifespan of the components (see the Technical Data chapter).

 Always use EMS original parts. Using non-original components may damage the equipment, and practitioner or patient may be injured.

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## SAFETY PRECAUTIONS

EMS and the dealer of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 **Contraindication:** ultrasonic oscillations may prevent cardiac pace makers and defibrillators from functioning properly. Therefore, we recommend that patients with a cardiac pace-maker or a defibrillator should not be treated with this product.

 **Use for the intended purpose only:** before using the product, make sure that you have studied the operating instructions. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 **Risk of explosion:** do not use this product in the presence of inflammable anaesthetics or gases.

 This product must be used only by trained and qualified personnel.

 **Check before each treatment** the contents of the bottle as well as the liquid present in the circuit.

 During treatment, avoid touching the patient with the handpiece, as the heat from the handpiece can be uncomfortable for the patient.

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 Clean, disinfect and sterilize the different pieces and accessories of this product before each use. Please refer to the information provided in the operating instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

 This product has been investigated with regard to safety from electrical shock and fire hazard. ETL (Electrical Testing Laboratories) has not investigated the physiological effects.

 Always examine the product for damage before commencing treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original EMS spare parts and accessories only.

 Have this product exclusively repaired by an approved EMS repair center.

 Use exclusively the external power supply (ref EK-357) delivered with the EMS unit.

 Use an approved pressure regulator, which is in compliance with the relevant national requirement. Limited pressure to 0.5MPa.

 Use exclusively the pedal (ref EK-346) delivered with the EMS unit.

 Certain types of mobile telecommunication equipment could potentially interfere with this product. The separation distances recommended in the "Electromagnetic compatibility" manual should be taken into account.

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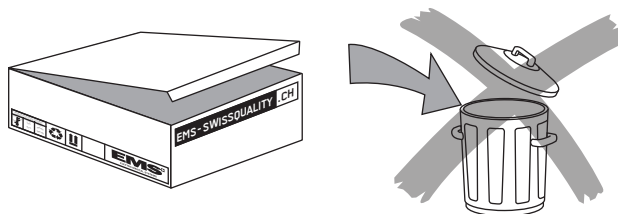
 Keep the original packaging until the product is to be disposed of permanently. It can be used for shipping or storing at any time.

Should you wish to put your product out of use for an extended period of time:

- Proceed as described in the "Cleaning, disinfecting and sterilizing" chapter,
- Proceed to a washing of the circuit without bottle to empty it,
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data" chapter.

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## DISPOSAL

The product must not be discarded in domestic household waste.

Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

Waste Electrical and Electronic Equipment belonging to customers located in the European Union may be shipped to EMS for recycling in accordance to the WEEE regulations. The costs of recycling, exclusive of shipping fees, are covered by EMS.

## WARRANTY

Damage due to non-adherence to the operating instructions or resulting from the wear out of parts are excluded from warranty.

 The warranty of your product will be cancelled if you try to open it.

## ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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## EMS SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or to an authorized EMS Repair Center.

In the case of non-authorized repairs or damages due to non-adherence to the operating instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

**Risk of transmitting micro-organisms!** To protect the personnel of the repair center and for safety reasons during transport and shipment, the product and the accessories returned to the factory for repair or revision must be cleaned, disinfected and sterilized in accordance with the operating manual. Repair can be refused for product or accessories received in a contaminated condition.

When sending your product directly to the approved EMS repair center, please include the name of your dealer. This simplifies processing.

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## TECHNICAL DATA

### Description

Manufacturer	EMS ELECTRO MEDICAL SYSTEMS SA, CH-1260 Nyon, Switzerland
Models	PM100 & PM200
Classification EN 60601-1	- Class II - Applied part, Type BF - IP 20, unit - IP X1, foot switch
Classification 93 / 42 EEC	Class IIa
Operating mode: With irrigation Without irrigation	Continuous operation Duty cycle 10% during 10 min. maximum
Power Supply: External voltage unit	100-240 VAC / 50-60 Hz 30 VDC
Power consumption	30 Watts
Ultrasonic specifications: Maximum output Frequency range	8 Watts 24-32 kHz
Weight	PM100: Approx. 0.6 kg PM200: Approx. 1 kg
Dimensions in mm (H x W x D)	PM100: 92 x 168 x 144 PM200: 92 x 150 x 215
Operating conditions	+10 °C to +40 °C 30% to 75% relative humidity Altitude maximum 3000m
Storage and transport conditions	-10 °C to +40 °C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure

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## TROUBLESHOOTING

Symptom	Solution
The LEDs of the power supply do not illuminate when the unit is switched on	<ul style="list-style-type: none"><li>• Check that the mains cord of the power supply is properly connected to the mains supply</li><li>• Check the power supply of the dental office</li><li>• Turn the power knob of the unit to ON and check if the LED is lit</li><li>• Replace the power supply</li><li>• Send the unit and the accessories to an approved EMS repair center</li></ul>
No irrigation flow when the foot switch is pressed	<ul style="list-style-type: none"><li>• Turn the liquid flow rate knob</li><li>• Make sure that water comes in external device for the PM100</li><li>• Make sure that the bottle is properly installed for the PM200</li><li>• Disconnect the ultrasound handpiece and check that the liquid exits from the cord</li><li>• Use compressed air (max. 3 bar) and check that air comes out of the handpiece</li><li>• If the air does not come out of the handpiece send to an approved EMS repair center</li><li>• Check the connection of the foot switch</li><li>• Replace the foot switch</li><li>• Replace the filter for the PM100</li><li>• Replace the pump for the PM200</li><li>• Send the unit and the accessories to an approved EMS repair center</li></ul>
No ultrasound oscillations when the foot switch is pressed	<ul style="list-style-type: none"><li>• Check that the handpiece is connected to the handpiece cord</li><li>• Control that the instrument is tight</li><li>• Check that the instrument is not cracked</li><li>• Check the connection of the foot switch</li><li>• Replace the foot switch</li><li>• Send the unit and the accessories to an approved EMS repair center</li></ul>
Ultrasound power decreasing or inadequate	<ul style="list-style-type: none"><li>• Check that the selector is beyond the 4-position</li><li>• Control that the instrument is tight</li><li>• Check the wear of the instrument and replace if necessary</li><li>• Replace the handpiece</li><li>• Send the unit and the accessories to an approved EMS repair center</li></ul>
Water leaks between the handpiece cord and the handpiece	<ul style="list-style-type: none"><li>• Replace the O-ring on the handpiece cord</li><li>• Replace the handpiece</li><li>• Send the handpiece to an approved EMS repair center</li></ul>
<i>Only PM200:</i> Water leaks from the bottle or the device	<ul style="list-style-type: none"><li>• Check the O-ring of the bottle cap</li><li>• Replace the bottle</li><li>• Send the handpiece to an approved EMS repair center</li></ul>

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© Copyright EMS SA
EMS Italia S.r.l.
EMS Corporation